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## Instruction for use seat stick

### Article ES 851

Please read these instructions for use thoroughly, carefully follow the details and keep them for later use.

#### Purpose

The seat stick serves as the seat support and can be used as a walking aid when folded.

#### 1. General product information

The Stick column is characterized by the following features:

- Three food alloy seat.
- Lightweight and robust with plastic seat.
- Folded also be used as a walking aid.

#### 2. Unfolding the seat stick

- Pull apart the seat stick at the handle and the seat.
- Then, turn down the seat.

#### 3. Material

Light alloy, seat plastic

#### 4. Cleaning

Wipe with a damp cloth and possibly with a mild household cleaner and let dry.

#### 5. Maintenance

- The seat stick is maintenance free.
- Should the seat stick have defects, contact your specialized dealer or local service for replacement.

#### 6. Disposal

Please inquire with your specialized dealer about the regional waste disposal.

#### 7. Warranty and Liability

You have purchased a high-quality Ossenberg product. If you have a justified complaint, despite highest quality standards in our manufacturing, we are entitled, at our discretion – within a 12 month warranty period - to repair the defective product or to replace it with a non-deficient one.

Please note that for our products, especially custom products, stricter legal regulations do apply. We cannot assume liability for damages resulting from modifications made to our products.

Not covered by the warranty are damages resulting from ordinary wear and tear or inappropriate use. Wearing parts are excluded from warranty.

For the entire range of Ossenberg products original spare parts and accessories are available. For spare parts, accessories and repairs, please contact your specialized dealer.

#### 8. Technical Data

Article	Material	Seat height	Total length (folded)	Weight	Max. Body Weight
ES 851	Light alloy, seat in plastic	57 cm	87 cm	0,80 kg	100 kg

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From us you will receive only products in accordance with the guidelines of the Medicine Devices Act.

We are certified according to DIN EN ISO 13485.

DIN EN ISO 13485  
 REG.-NR. MQ1 0115318

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**Ossenberg GmbH**