FREEBREE FFP2^{R-K2}

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FILTERING HALF MASK FOR RESPIRATORY
PROTECTION AGAINST SMALL PARTICLES

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FILTERING HALF MASK FOR RESPIRATORY PROTECTION AGAINST SMALL PARTICLES. TYPE ACCORDING TO EN 149: 2001 + A1: 2009: FFP2 MODEL: R-K2; 5 - LAYERS. THE PROTECTIVE DEVICE COMPLIES WITH THE EUROPEAN STANDARD EN 149:2001+ A1:2009.

EN / PRODUCT INFORMATION

THE NOSE AND MOUTH COVERING FILTERING HALF MASK IS INTENDED TO PROTECT THE HUMAN RESPIRATORY SYSTEM FROM THE SMALL PARTICLES. WASH YOUR HANDS THOROUGHLY BEFORE PUTTING ON THE FILTERING HALF MASK. PLACE THE LOOPS BEHIND BOTH EARS SO THAT THE NOSE CLIP IS ON TOP. SQUEEZE THE NOSE CLIP TO FIT THE SHAPE OF THE NOSE. PUT ON THE FILTERING HALF MASK SO THAT IT FITS WELL TO THE FACE, COVERING THE NOSE, MOUTH AND CHIN. PROTECTIVE DEVICE IS LESS EFFECTIVE IF APPLIED ON AN UNSHAVEN FACE AND IF THERE IS A LOT OF COSMETICS ON THE FACE. THE USE OF THIS PROTECTIVE DEVICE DOES NOT PROTECT AGAINST HARMFUL GASES, VAPORS AND AEROSOLS. THIS PROTECTIVE DEVICE DOES NOT GENERATE OXYGEN AND IS THEREFORE NOT SUITABLE FOR USE IN AN OXYGEN-FREE ENVIRONMENT, UNDERWATER OR IN CASE OF FIRE EVACUATION. THE USE OF THIS FILTERING HALF MASK IS NOT RECOMMENDED IN CASE OF RESPIRATORY PROBLEMS DURING THE USE. IF DURING USE YOU NOTICE THAT THE FILTERING HALF MASK IS DAMAGED OR WET, REPLACE IT. THIS PROTECTIVE DEVICE IS DISPOSABLE AND CANNOT BE REUSED.

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