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Bild 1

Adjusting the walking aid

Warning! Please contact your specialist dealer to ensure proper functional adjustment! Incorrectly adjusted walking aids may restrict the functionality of the walking aid or lead to overloading or incorrect loading of the patient! The walking aids are hardened over to you as a patient in a proper condition, whereby they are individually adjusted to your body by your specialist dealer or health care specialist. If you have any doubts as to whether this is the case, please contact the responsible specialist personnel. The walking aid can only offer you the relief you desire if it is optimally adapted to your body size. **Adjustment:** The union nut must be loosened by approx. half a turn anticlockwise prior to adjustment (Fig. 1), Unlock the height adjustment by pushing out the clip. Adjust the length of the walking aids to your body size by pulling the support tubes out or pushing them together. Lock the height adjustment once again by aligning the tubes as such, that the two locking holes are on top of each other and reinsert the clip. **Following adjustment, the union nut must be tightened again. Caution:** Risk of falling due to walking aids buckling when the height adjustment is not locked correctly! Check the correct locking after each adjustment by attempting to twist or move the walking aids!

Bild 2

Use

There are many different ways to use the five-foot crutch and it all depends on the individual restrictions of the walking ability. Seek instruction in the use of the walking aids from your specialist dealer or health care specialist! Walking aids must not be used in water. Any penetrating water can lead to corrosion forming and consequently to an increased risk of breakage. Avoid prolonged contact of the walking aids with moisture. The walking aids must be dried appropriately when they come into contact with water. The correct handling of the walking aids takes some practice. We therefore recommend that you seek advice from a specialist dealer or health care professional. Always keep the walking aids on the side opposite the injured leg or the leg that needs to be protected! Guide the walking aids close to the body, not at an angle facing outwards (Fig. 2)! You might fall otherwise! During use, grasp the handle(s) of the walking aid(s) completely and firmly with your hand! Always place the walking aids as straight as possible on the ground so that you do not slip away! Find chairs with armrests to sit on. You will then be able to support yourself on the armrests when standing up! Wear shoes that are as comfortable as possible with flat soles so that your leg can better withstand additional strain! **If you do fall:** If possible, let the walking aid fall to the side so that you do not fall onto it! Have someone accompany and support you if possible, so that they can catch you if you fall! Try and arrange for qualified personnel to instruct you in climbing stairs with a walking aid!

fig. 2

Precautionary measure

Due to the fact that a walking aid places an unusual amount of strain on individual parts of the body! This may result in the following undesirable side effects: The hands may encounter pressure or chafing! Keep in mind that this can lead to pain and overexertion at the beginning or during prolonged exertion! It is important to take sufficient breaks!

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Due to the fact that a walking aid places an unusual amount of strain on individual parts of the body! This may result in the following undesirable side effects: The hands may encounter pressure or chafing! Keep in mind that this can lead to pain and overexertion at the beginning or during prolonged exertion! It is important to take sufficient breaks!

Storage and care

Never store the walking aids in the vicinity of heat sources such as furnaces, heaters or in places susceptible to heat in a car! Do not expose walking aids to sunlight for prolonged periods of time! Strong sun or heat damages the plastic! **The rubber buffer is a wearing part and must be replaced at regular intervals.** Walking aids must only be used again after an extended period of non-use following prior inspection by qualified personnel! **Warning!** Never clean handles and rubber caps using oily substances! There is a risk of reagent! Rutschgefahr! Reinigen Sie die Gehhilfen bei Verschmutzungen gegebenenfalls mit einem milden Reinigungsmittel und einem weichen Lappen. Verwenden Sie niemals losmittelschädigende Reiniger, da diese den Kunststoff beschädigen! **Hinweis:** Wird eine Desinfektion notwendig, ist ein geeignetes Desinfektionsmittel zu verwenden. Bitte beachten Sie bei der Anwendung die Herstellerangaben!

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Instructions for Use: Five-foot crutch

EN

Intended use and indication

Our walking aids are designed for people with significant mobility impairment with impairment to movements, compromised balance and stability with sufficiently intact walking ability and coordination. They are used to maintain, promote and stabilise walking or to partially relieve the strain on the lower limbs. The walking aids can be used both indoors and outdoors. **It is imperative that the safety instructions are also observed!**

Contraindication

The use of walking aids is unsuitable if there is a loss of limb in both arms or joint contractures/joint damage in both arms.

General safety information

Do not hang any bags, pouches or other objects on the walking aid! Walking aids may under no circumstances be used for any other purpose than that for which they are intended, e.g. for lifting or moving objects or as a tool for switching light switches or pressing buttons on lifts! It is essential that the ground has a good grip to ensure a secure hold of the walking aids! Avoid wet floors or surfaces such as gravel, sand, foliage, ice or snow! **Please pay attention to stumbling points** such as unevenness, holes in the floor, loose floor coverings or edges, because the walking aid can unexpectedly slip away or sink! Extra caution is recommended when lighting conditions are insufficient, since irregularities in the ground can then only be seen with difficulty! The holes for adjusting the walking aid are under a certain amount of wear. It is important to ensure that they are not worn out, as this would prevent secure locking. The plastic clip must have a positive fit when inserted (Fig. 1)! Prior to each use, make sure that all screw connections are firmly tightened!

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