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EN- Position the rubber tip at the bottom. Keep the rod vertical for the rest of the assembly.

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EN- Carefully slide the handle along the rod to the correct height.

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
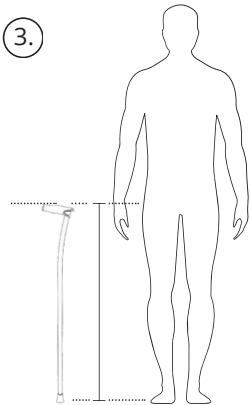
EN- Check the height at which the handle will go.

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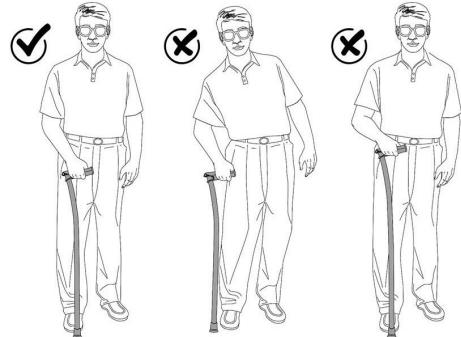
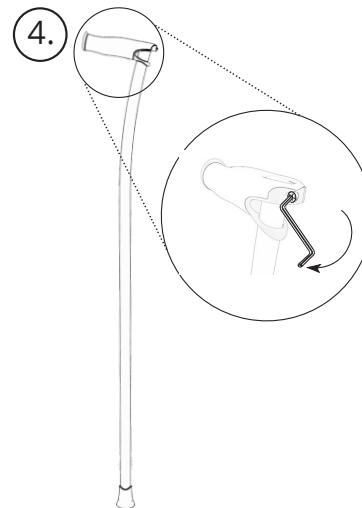
EN- Tighten the screw with the Allen key until resistance is met. Make another half turn without forcing.

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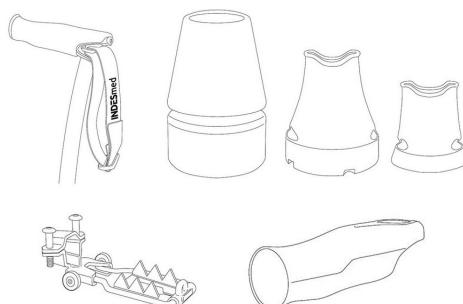EN - INDES MED ADVICE FOR MOUNTING

1- To facilitate the sliding of the handle along the rod:

- Open the screw as much as possible, without dropping the nut.
- If necessary, use a coin to enlarge the opening.
- Apply neutral liquid soap on the inside of the metal ring.

2- If the handle is not fixed, check if it is mounted in the correct position. Make sure you tighten the screw correctly and do not lose the nut.

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EN - INDICATIONS

Aid and assistance for walking in general. People with neurological pathologies or diseases that

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require light crutches. People with lack of balance. In cases where, in addition to the above, a reduction of pain in wrists, palms of the hands, shoulders is required, and reduction of tendinitis in general due to its use.

EN - CONTRAINDICATIONS

Not indicated for children under 5 years of age.

EN - CARE INSTRUCTIONS

Wash with neutral products and water. Periodically check all parts and replace if necessary.

EN - WARNINGS AND CAUTIONS

- Not for use by persons weighing more than 200kg for canes with carbon fiber tubes or 140kg in case of aluminum tubes.

- In case of observing a malfunction of the device, notify the manufacturer as soon as possible: info@indesmed.com. The manufacturer will proceed accordingly. Any serious incident, death or injury must be notified to INDES MEDICAL S.L. and to the Competent National Authority of the country of residence of the user/patient.

- At the end of its useful life, dispose of the device at the recycling points.

EN - WARRANTY

INDES medical S.L. guarantees all its products for 24 months, as long as they have not been manipulated or altered in their original configuration. The carbon fiber tubes have a 48 months warranty. The products are not guaranteed if, due to misuse, deficiencies or breakage of any kind, their characteristics are altered. Tips and handle covers are considered as fungible material exempt from warranty.

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