DE

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EN

Directions for use and safety instructions

The reacher is making it easier for people with limited arm function, to pick up and put down almost all kinds of items.

1. General product information

The reacher is characterized by the following features:

- ergonomically designed grip
- rubberised surface for safe grip of small parts
- integrated magnet to lift small metallic objects
- hand loop for safe use

2. Use

- hold the reacher with one hand at the handle
- pressing the leverage the lever mechanism is activated and the pincer closes
- releasing the lever mechanism and the pincer opens

Note: Use carefully to avoid damage to objects and persons.

3. Material

The handle and the pincer are made of an acrylonitrile-butadiene-styrene copolymer (ABS plastic), the tube is made of aluminium, the hand loop is made of polypropylene (PP) and the anti-slip pad is made from rubber.

4. Cleaning

Wipe with a damp cloth. Only use mild household cleaner and let air dry.

5. Maintenance

- The reacher is maintenance free.
- If malfunctions or damage occur, please contact your local dealer.

6. Disposal

Please talk to your dealer about local disposal regulations.

7. Warranty and Liability

You have purchased a high-quality product. If you have a justified complaint, despite highest quality standards, we are entitled, at our discretion – within a 12 month warranty period - to repair or replace the product. We don't assume liability for damages resulting from modifications made to our products. Not covered by the warranty are damages resulting from ordinary wear and tear or inappropriate use. Wearing parts are excluded from warranty.