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## General product information

The anti-slip, low-abrasion adhesive buffers are mounted on walking aids with an outer diameter of 16, 18, 20 or 22 mm and give the user of the walking aid a great deal of stability, especially on wet surfaces. The two-part construction of the adhesive buffers absorbs shocks and, thanks to the joint plug, allows the user to stand securely, even if the walking aid is placed at a slight angle.

## Application and safety notices

Prior to each use, check the functionality of your walking aids and ensure the tight fit of the adhesive buffers. Regularly check the adhesive buffers for damage or wear and replace them if necessary.

For safety reasons, the top and the bottom part were joined mechanically. If the bottom part is worn down, it may not be detached from the top part and replaced separately. The bottom part may become darker over time due to incident light radiation. This does not impair the quality in any way. The top part is highly resistant against ageing.

The danger of slipping or falling down -especially on slippery or wet surfaces- can never entirely be ruled when using walking aids.

Place the walking aids as vertical as possible on the surface when walking or not too far away from the body. Setting the walking aid down at a very slanted angle increases the risk of fall.

## Material and Design

**Material:** Plastic

**Design:** Two-part structure consisting of a soft bottom part and a hard top part with an integrated metal disk to prevent the walking aid tube from breaking through.

## Storage and Cleaning


The adhesive buffers should be stored in a cool and dry place, out of direct sunlight. Cleaning can be carried out with commercially available cleaning product.

## Warranty and liability

You have purchased a high-quality Ossenberg product. If, despite the highest quality standards, a justified material defect should occur, Ossenberg has the right, within the framework of a 12-month warranty, to repair the defective item or to replace it with a defect-free subsequent delivery, at its own discretion. We cannot accept any liability for damage caused by modifications to our products. Not covered by the warranty are defects that are due to wear or improper handling. Wear parts are excluded from the warranty.

## Disposal

Please ask your specialist dealer about the regional disposal regulations.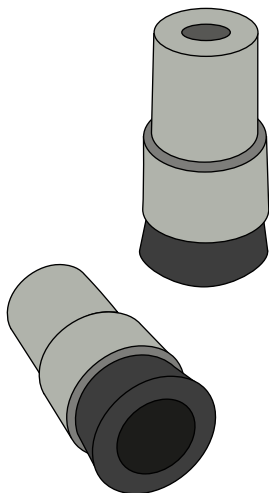

## Technical data

Art. Nr.	for tube- $\varnothing$
----------	-------------------------

93816P	16 mm
--------	-------

93818P	18 mm
--------	-------

93820P	20 mm
--------	-------

93822P	22 mm
--------	-------