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General product information

The umbrella TWIN is a combination of umbrella and walking stick. A small round button locks the walking stick into the umbrella tube. If required- a slight touch of the button quickly separates umbrella and walking stick.

Instructions of use

If the umbrella and walking stick are needed separately, proceed as follows:

1. Open the umbrella, just as you are accustomed to, by sliding it up to the stop. The umbrella is equipped with a self-locking semi-automatic.
2. When open, you see a small round button at the lower end of the handle. With one hand, hold the umbrella handle while simultaneously pushing the button, you can now with the other hand move the handle and pull out the walking stick of the umbrella tube. Put up the umbrella or set aside closed. Both the umbrella and the walking stick can now be used individually. This unique combination may replace an additional walking stick.
3. If the walking stick should again be join with the umbrella, slide it into the umbrella tube without again pushing the round button.
4. Eventually turn the walking stick slightly until it snaps into the hole of the umbrella tube.

Adjust your correct hight of the walking stick with the push-button:

1. Unlock the height adjustment by pushing the button.
2. Adjust the length of the walking stick to your correct body height and lock the height adjustment again by lining up the tubes so that two locking holes align. The push button must audibly snap in the hole.

- The walking stick and umbrella are both equipped with a rubber tip to prevent slipping on smooth surfaces.
- Indoors, there is of course no need of an umbrella.
- The umbrella has a special fiberglass rib construction. If reversed at a strong wind gust, the flexible rib constructio can easily be folded back to its normal position without twisting or damaging the umbrella.

Suggestion:

- After getting wet in rain, dry the umbrella in open position (not in the sun)!
- The rubber tips of the walking stick and umbrella should be replaced when worn down. You can buy spare tips at your local dealer.

Storage and care

Never store the umbrella near heat sources such as ovens, heaters or in the car on the shelf! Strong sun or heat can damage the material! **Warning!** Never clean handles and rubber capsuels. Danger of slipping! In case of soiling, clean with a mild detergent and a soft cloth, if necessary. Do not use cleaners containing solvents, as they may damage the material!

Warranty and liability

You have purchased a high-quality Ossenberg product. If, despite the highest quality standards, a justified material defect should occur, Ossenberg has the right, within the framework of a 12-month warranty, to repair the defective item or to replace it with a defect-free subsequent delivery, at its own discretion. We cannot accept any liability for damage caused by modifications to our products. Not covered by the warranty are defects that are due to wear or improper handling. Wear parts are excluded from the warranty. Only original spare parts and accessories may be used! For spare parts, accessories and repairs, please contact the specialist trade.

Disposal

Please ask your specialist dealer about the regional disposal regulations.

Technical data

Article no.	Variant	Design
3030	Big variant	Check brown
3033	Big variant	Check red / blue
3034	Big variant	black
3036	Big variant	Check blue / green
3038	Big variant	Paisley bordeaux
3039	Big variant	Design houndstooth
3060	Big variant	Check black / blue / red / white
3130	Small variant	Check brown
3133	Small variant	Check red / blue
3134	Small variant	black
3136	Small variant	Check blue / green
3138	Small variant	Paisley bordeaux
3139	Small variant	Design houndstooth
3160	Small variant	Check black / blue / red / white

Umbrella	Big Variant	Small Variant
Special light-metal tube	Ø 16 mm	Ø 16 mm
Weight	605 g	568 g
Umbrella-Ø	ca. 105 cm	ca. 105 cm
Hight adjustable (Umbrella incl. Walking Stick)	87 cm- 95 cm	80 cm- 83 cm

Walking stick	Big Variant	Small Variant
Tube	Ø 14 mm	Ø 14 mm
Weight	242 g	215 g
Hight adjustable	5-times by push button 85 cm – 93 cm	3-times by push button 78,5 cm – 81 cm
Fritz grip length	12,5 cm	12,5 cm
Max. user weight	100 kg	100 kg

Accessories	
Article no. 969P	Rubber tip for umbrella, inner- Ø 16 mm
Article no. 3086P	Rubber tip for walking stick, inner- Ø 10 mm

You can find additional accessories under www.ossenberg.com

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With us you receive products according to the guidelines of the Medical Devices Act. Furthermore, Ossenberg GmbH is certified according to DIN EN ISO 13485.