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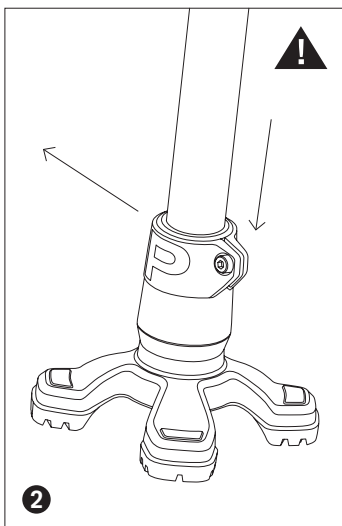
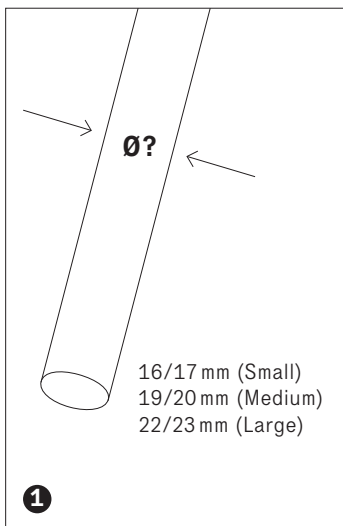
EN INSTRUCTION MANUAL SAFETY FOOT

SAFETY FOOT
WALKING AID ATTACHMENT

WALKING AID ATTACHMENT ART. NO. SFG2SS / SFG2SM / SFG2SL / SFG2PS / SFG2PM / SFG2PL

Please keep this manual in a safe place for future reference.

1 Please measure the shaft diameter at the bottom of your walking aid.

2 Put the walking aid attachment

onto the crutch tube like this, so that the clamping lugs face towards the back of the clamp, viewed in the direction of movement. Tighten the screw with the Allen wrench supplied.

Make sure that the screw is tightened.

The product is only approved for its intended use. In the event of incorrect use the manufacturer accepts no liability whatsoever. Prior to each use, check that the walking aid attachment is functioning properly. Damaged and worn footpads should be replaced immediately by your specialist dealer. Unauthorised modifications to the product are not allowed and release the manufacturer from any liability and warranty claims by the user.

The danger of skidding or falling – especially on smooth or wet surfaces – can never be completely ruled out during use. When walking position the crutches as upright as possible and not too far away from the body. Positioning them at an angle increases the risk of falling.

Intended use The SAFETY FOOT was developed and manufactured with great care in Germany (100 %). Thanks to the foot's innovative geometry and the careful choice of materials, the SAFETY FOOT is at home on every surface. It is suitable for moving on wet, soft or uneven surfaces (for example sand, lawn, gravel, forest and dirt tracks).

Indications Where there is an indication for walking sticks or crutches to improve walking and standing safety under various ground conditions.

Contraindications Severe balance disorders or significant weakness of the lower extremities.

Cleaning Clean under running water (max. 40 °C) using a soft cloth and a pH-neutral, non-abrasive household cleaner. Rinse with clean water and allow to dry.

Materials/Disposal High-strength engineering plastics (e.g., polyamide, polypropylene, glass-fiber reinforced), stainless steel (screws, washers, nuts). The walking aid attachment is made entirely from recyclable materials. For disposal, please contact your distributor.

Service Life Recommended service life with proper use: up to 2 years. The actual service life depends heavily on usage, surface conditions, and load. With intensive use, earlier replacement may be required. When stored properly (dry, protected from light, at room temperature), the product can be stored indefinitely.

Maximum Permissible Load Up to 200 kg body weight.

Warranty/Liability Congratulations — you are the owner of a high-quality and highly innovative product from Nikon-Tec GmbH. Should a justified material defect occur, Nikon-Tec GmbH reserves the right, within a 12-month warranty period from the date of purchase, to either repair the defective product or replace it with a fault-free product at its discretion. No warranty is provided for wear parts. Serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority.

Original spare parts are available for the entire Nikon-Tec range. Your local dealer will be glad to help you further.

For users with visual impairments, this document is available as a PDF file for enlarged display on our website.

Manufacturer Address
Nikon-Tec GmbH
Bücklestraße 3
DE-78467 Konstanz

Issue Date and Version
01.01.2026 Version 2

REF Art. No.  Manufacturer  Date of Manufacture
LOT Batch Number **MD** Medical device **CE** CE marking
 Follow instructions for use **UDI** UDI marking, (01) GTIN (UDI),
(11) Production date YYMMDD, (21) Serial number or (10) LOT number

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